



HOW TO USE CLEAN SOFT EX

- Remove stained color
- Remove Candida

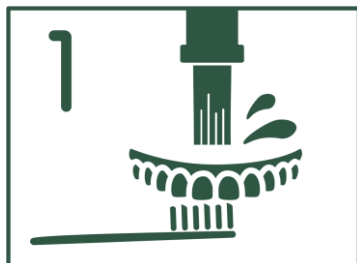

- 1 Remove any contaminants from oral appliances (dentures/mouthguards/orthodontic retainers) using a soft-bristled toothbrush under running water.

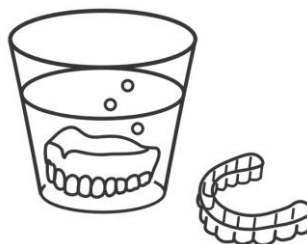

- 2 Shake the bottle once and add one dose of this product (approximately 1g) to a cup (approximately 150ml) of cold or lukewarm water (15-40°C) and stir well. Do not use with hot water (over 60°C).



- 3 Soak the entire oral appliance for at least one hour.



- 4 After cleaning, rinse thoroughly under running water using a soft toothbrush or similar tool.



KAMEMIZU CHEM.IND.CO.,LTD.

Address:17-16 TOYOSATO-CHO, NEYAGAWA, OSAKA 572-0071, JAPAN
TEL +81-72-826-7720 FAX +81-72-826-1299