

For cleaning denture plaque


# Clean soft

Denture cleaning and preservative powder for soft resin

## Instruction



1 Take out a spoon from the container.

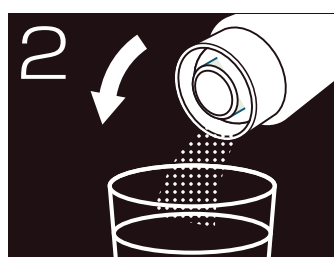

2 Pour tap water into a cup enough to soak a denture completely. Then put a spoon full of powder into the cup, and mix well.



3 Put the denture into the cup and leave it for overnight. As the powder easily gets moist, please tighten the lid firmly after use.



4 In the following morning, rinse the denture with running water thoroughly before use.  
\* It is not suitable for removing stained color of denture

- \* Avoid high temperature and humidity, and store the powder in a cool, dark place.
- \* Even if the mixed liquid contains precipitates or becomes clouded, it doesn't have influence on an effect, so please use it as it is.
- \* Please do not put the powder or the liquid into the mouth.
- \* Please place it out the reach of children.



**KAMEMIZU CHEM.IND.CO.,LTD.**

Address:17-16 TOYOSATO-CHO, NEYAGAWA, OSAKA 572-0071, JAPAN  
TEL +81-72-826-7720 FAX +81-72-826-1299  
E-mail : info@kamemizu.co.jp URL : <http://kamemizu.co.jp>