

# *Reduce use of denture adhesives*

*To make better-fitting dentures  
Developed over the years by visiting dentist*

**Resistant  
to dirt**

**Easy to  
trimming**

**Can use  
denture cleanser**



**New  
type**

**Advanced  
Tissue Conditioner**

**Keep softness  
High patient satisfaction**

**DENTURE SOFT  
EX**



Colors Pink/ White

Certification No. 304ADBZX00109000  
Short-term soft lining material (Medical device)




**KAMEMIZU CHEMICAL IND.CO.,LTD.**  
17-16 Toyosato-cho, Neyagawa, Osaka 572-0071, Japan

TEL +81-72-826-7720  
FAX +81-72-826-1299  
<https://kamemizu.co.jp>  
E-mail: info@kamemizu.co.jp

# Soft



Reduces the burden on the mucosa under the denture base.

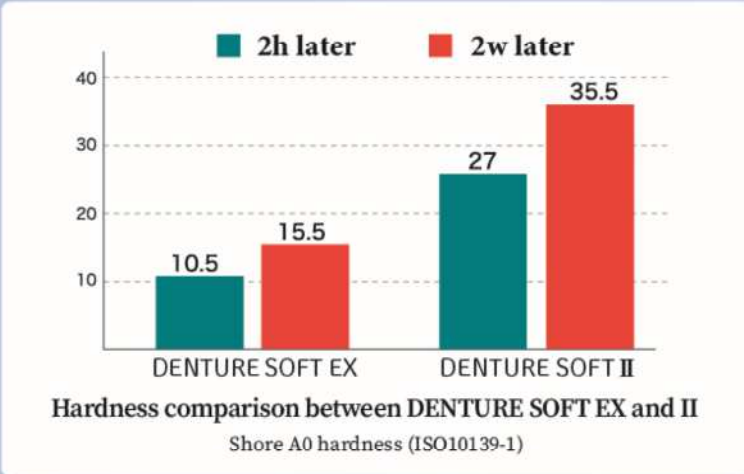

# Resistant to dirt



Commercially available denture cleanser "Polydent with Enzymes" (Haleon plc) can be used.

Note 1) Do not soak for more than 5 minutes until 1 week from lining.

Note 2) Do not use Polident other than "Polydent with Enzyme" or other commercial denture cleansers.

# Easy to handle

- Good powder-liquid blending and smooth mixing
- Carbide burs and big silicon points can be used



## Packaging

## DENTURE SOFT EX set

### Contents

Powder 110g, Liquid 100mL, powdermeasuring cup, mixing cup, dropper, spatula



### Colors

Pink · White

# DENTURE SOFT EX


## How to use

### 1 Adjustment



If necessary, the old mucous membrane surface of the denture base to be removed with a bur. Then clean and dry.

### 2 Measuring

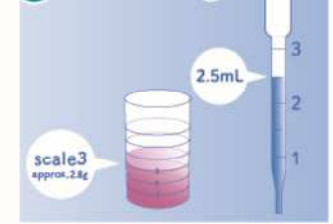

The standard powder/liquid ratio is 2.5 mL of liquid to measuring cup scale 3 of powder (approx. 2.8 g). For full upper dentures, double the amount of each.

### 3 Mixing



Mix the weighed powder and liquid for about 20 seconds. It will become syrup like in 1-2minutes after mixing.

### 4 Insertion



Apply the resin to the mucous membrane surface of the denture base. Make it evenly thick with a spatula, etc. and insert to the mouth. (Keep in mouth about 5-6 minutes)

### 5 Trimming



Remove excess material using scissors or a knife.

### 6 Completion



Available in 2 colors. (pink and white)

The liquid may separate or become cloudy at low temperature. In such a case, please stir strongly and warming to room temperature and use after confirm the liquid returns transparent.

Note: Please read the instructions for use carefully before use.

## Related Products

Adhesive and Surface smoothing material  
New Top Coat



Used exclusively with acrylic soft reline materials as an adhesive.

Note: Do not use denture cleanser when "New Top Coat" is used as a surface smoothing on the mucous membrane surface of the denture base.